



# GoCheck Kids. QUICK START GUIDE

## GOCHECK MULTISCREENER- HEARING

### Step 1: Set Up Your GoCheck Kids Account

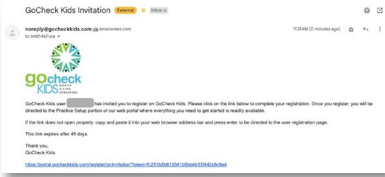

Find the "GoCheck Kids Invitation" email in your inbox and click the invite link to the web portal.

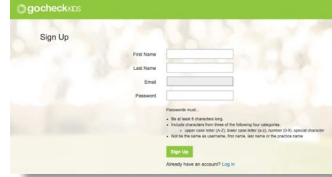

Create a user account and password. This will be your login for both the app and the web portal.

### Step 2: Set Up the Device



Connect to Wi-Fi through the iPhone settings. Set up a device passcode.




Tap the GCK Hearing app icon.




Log in with your user account.

### Step 3: Enter a Patient Into the Queue

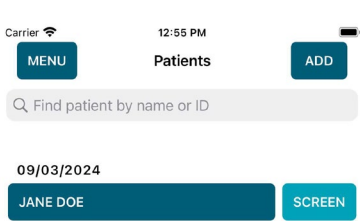

Tap "Add" at the top right of the screen

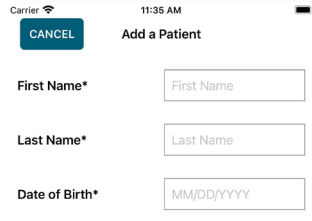


Enter the Visit Date, First Name, Last Name, and DOB of the patient.


## Step 4: Screen a Patient




Find the patient and tap Screen.

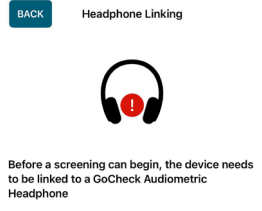

Plug in and link headphones.

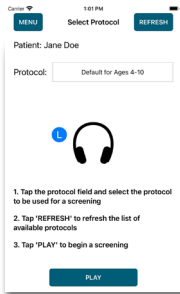


Choose a protocol and press Play.

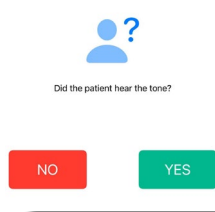

Record results for each tone.

## Step 5: View and Export Results

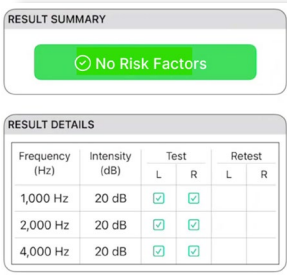

Immediately view the results. The results can be exported via fax or printing.

Note: Ensure you are connected to WiFi after screening to upload results.

Scan the QR Code or Visit the Link to Access the User Manual  
<https://learn.gocheckkids.com/multiscreener-user-manual/>



Gobiqity, Inc.  
 25020 Avenue Stanford  
 STE 100  
 Santa Clarita, CA 91355

©2024 Gobiqity, Inc. Gobiqity, the Gobiqity logo, GoCheck Kids, and the GoCheck Kids logo are trademarks of Gobiqity, Inc. All other trademarks are the property of their respective owners.  
 Doc. #9000-013 Rev A



For additional support, contact us at

[support@gocheckkids.com](mailto:support@gocheckkids.com)  
 or call 866-633-9243