





Visual Acuity Instructions for in-office use


1. First, log out and log back in to the app to refresh.
2. For this test, you will need a tester to hold the device, an occluder to cover the patient's eye, and a tape measure.

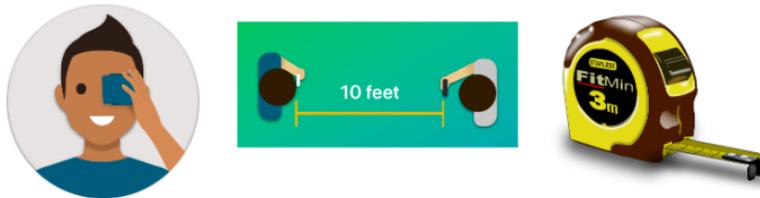

3. Next, measure 10 feet between the phone and the patient. Add patient information in the app and select "Start 10 Feet VA Test". Follow the instructions in the app to continue.
4. Give the occluder to the patient. To begin the test, turn the device screen to the patient.

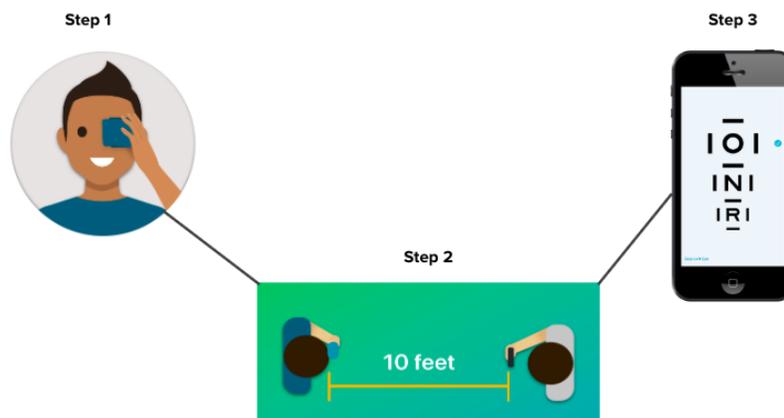

5. Ask the patient to read the letters inside the crowding bars from top to bottom. Make sure to remember the patient's responses.

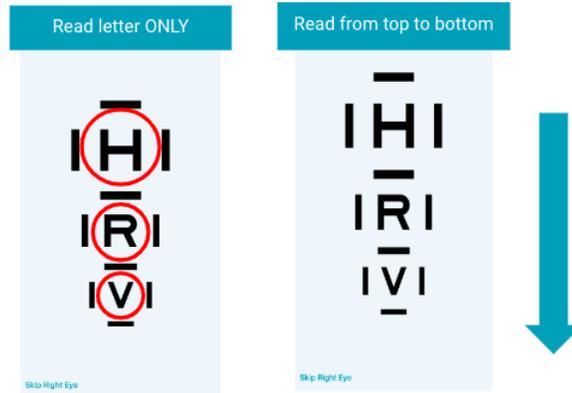

6. To record, tap once for the correct result and twice for the incorrect result. Record all letters to advance.



7. The result report is then immediately available in the app and uploaded to the web portal.

No risk factor identified		Right eye @ 10 feet	
8/26/2020 04:30 PM		20/20 = LogMAR: 0.00	
Visual Acuity Result		False Negatives: 0/7 100.0% test reliability	
		False Positives: 0/0 N/A	
		Overall: 100.0% test reliability	
		Duration: 5 seconds	
Right Eye	Left Eye	20/63 P	
20/20	20/20	20/50 P	
LogMAR: 0.00	LogMAR: 0.00	20/40 P	
		20/32 P P	
		20/25 P P	
		20/20 P P P P P	
Threshold: 20/40 for age 4		Left eye @ 10 feet	
Optotype: HOTV		20/20 = LogMAR: 0.00	
Distance: 10 feet		False Negatives: 0/7 100.0% test reliability	
Vision Correction: none		False Positives: 0/0 N/A	
Scroll to view details		Overall: 100.0% test reliability	
		Duration: 5 seconds	