



# GoCheck Kids QUICK START GUIDE

## GOCHECK MULTISCREENER- VISION

### Step 1: Set Up Your GoCheck Kids Account

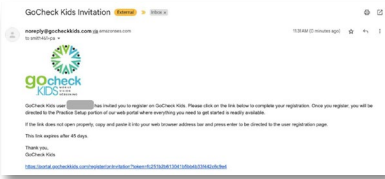

Find the "GoCheck Kids Invitation" email in your inbox and click the invite link to the web portal.

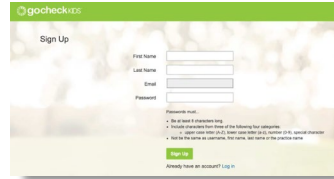

Create a user account and password. This will be your login for both the app and the web portal.

### Step 2: Set Up the Device



Connect to WiFi through the iPhone settings. Set up a device passcode.



Tap the GoCheck Kids app icon.




Log in with your user account.

### Step 3: Enter a Patient Into the Queue




Choose "Add" at the top right of the screen

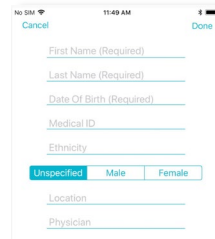


Enter the Visit Date, First Name, Last Name, and DOB of the patient.

## Step 4: Screen a Patient

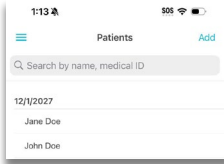


Select the patient for screening

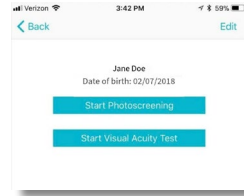

Start the screening test.



Follow the on-screen prompts to complete the screening test.

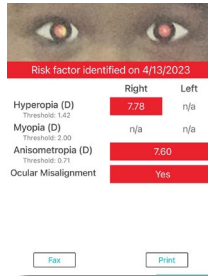

Immediately view the results. The results can be exported via fax or printing.

Note: Connect to WiFi after screening to upload results.

## 3 Tips for Screening Success

1.



For best results with photoscreening, use dim, indirect lighting.

2.



Ensure proper test distance: 40 inches for photoscreening, or 10 feet for visual acuity testing.

3.



For best results with visual acuity screening, don't obscure the screen of the device.

Scan the QR Code or Visit the Link to Access the User Manual  
<https://learn.gocheckkids.com/multiscreeener-user-manual/>



Gobiquity, Inc.  
 25020 Avenue Stanford  
 STE 100  
 Santa Clarita, CA 91355

©2024 Gobiquity, Inc. Gobiquity, the Gobiquity logo, GoCheck Kids, and the GoCheck Kids logo are trademarks of Gobiquity, Inc. All other trademarks are the property of their respective owners.  
 Doc. #9000-009 Rev A



For additional support, contact us at

[support@gocheckkids.com](mailto:support@gocheckkids.com)  
 or call 866-633-9243