

GoCheck Kids. QUICK START GUIDE

GOCHECK AUDIOMETRIC HEADPHONES

Step 1: Open GCK Hearing App



Connect to WiFi through the iPhone settings. Set up a device passcode.




Tap the GCK Hearing app icon.



Log in with your user account.

Step 2: Pair Headphones

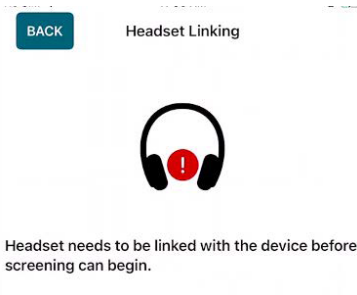

Plug headphones into the device. You will see instructions for linking headphones when you open the hearing app.




Locate the QR code and serial number on the headphones band.

1. To link with QR code by pressing "Automatic Scan"
2. To link with serial number by pressing "Manual Entry"

AUTOMATIC SCAN

MANUAL ENTRY

You have two choices: Automatic Scan, which allows you to scan a QR code, and Manual Entry, which allows you to enter a serial number.

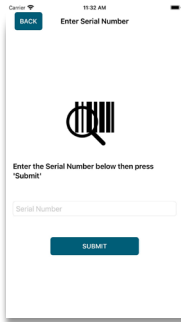

or



Enter the 10-digit serial number below the QR code.

Automatically scan the QR code.

Step 3: Note the Calibration Date



Your headphones require annual calibration. Locate the UDI label on the headband adjacent to the QR code. The UDI number contains the calibration date in YYYYMMDD format in the last line, here it is August 5, 2025.



This product generates stronger permanent magnetic fields that could cause interference to cardiac pacemakers, implanted defibrillators (ICDs) and other implants. Always maintain a distance of at least 3.94" (10 cm) between the product component containing the magnet (the earmuffs) and the cardiac pacemaker, implanted defibrillator, or other implant.

Scan the QR Code or Visit the Link to Access the User Manual
<https://learn.gocheckkids.com/multiscreener-user-manual/>

Gobiqurity, Inc.
25020 Avenue Stanford
STE 100
Santa Clarita, CA 91355

2024 Gobiqurity, Inc. Gobiqurity, the Gobiqurity logo, GoCheck Kids, and the GoCheck Kids logo are trademarks of Gobiqurity, Inc. All other trademarks are the property of their respective owners.
Doc. #9000-010 Rev A



For additional support, contact us at

support@gocheckkids.com
or call 866-633-9243